



User Manual

ProReset Vista





Congratulations on choosing one of the most innovative hyperbaric chambers on the market. Now that you have all the equipment to begin your hyperbaric journey, it is extremely important to read this user manual before operating your chamber.

SAFETY FIRST

At ProReset, safety is our first priority. It is our policy and practice to ensure that ProReset chambers have the highest quality, comfort, and safety.

We have implemented 6 safety features:

- ✓ Solid Windows
- ✓ Internal and external bidirectional communication and control system
- ✓ Automatic air pressure control and ventilation system
- ✓ Air pressure switching device, four adjustable pressures
- ✓ Real-time oxygen concentration display function
- ✓ Automatic disinfection and deodorization system



Contents

Preface

1. Precautions	5
2. Specification structure.....	7
Chamber.....	7
Air compressor.....	8
3. Assembly test.....	10
Unpacking.....	10
Compressor assembly.....	10
Pressure regulation.....	11
How to adjust the seat.....	12
How to enter and exit the chamber.....	12
Use of communication equipment.....	14
Use of the Chiller.....	15
Console scheme.....	15
4. Maintenance.....	16
Chamber maintenance and cleaning.....	16
Maintenance and cleaning of Air compressor.....	16
5. Reminders.....	17
6. ProReset Warranty Policy.....	19
7. Troubleshooting.....	22



Preface

Thank you for purchasing a ProReset hyperbaric oxygen chamber (hereinafter referred to as "this product").

The chamber body of this product is made of durable material, such as a stainless steel, pressurized with clean air that has been filtered through multiple layers, and the pressure in the chamber can increase to 2.0 ATA or 100 kPa. It offers safety, reliability, and convenient operation. It can be widely used by families in home settings, wellness and community service centers, sports venues, gyms, spas, beauty and health care centers, physiotherapy centers, ships, and other public environments.





Precautions

To ensure safety, please read the following precautions before use.

Important Safety Considerations

Do NOT use the chamber if you have the following:

- Pregnant
- Common colds or flu-like symptoms
- Recent alcohol consumption
- Blocked ear canals
- Blocked sinuses
- Severe emphysema
- Otic barotraumas
- Excessive CO₂ exposure
- Hypertension exceeding 160/100 mmHg
- Using a pacemakers, first check with the pacemaker manufactere

Warning:

- If an abnormal smell occurs during use, turn off the power and pull out the plug.
- Please make sure that the socket plug is fully inserted firmly.
- Please do not disassemble the machine at will.
- This product should be kept away from direct exposure to sun, water and fire sources.
- Children must be under the supervision of adults at all times.
- Please do not temper with the valves chamber.
- Please do not place heavy objects on top of the chamber or any accessory.
- Do not use if any wire is loose.
- Do not pull or twist the power cord forcibly.
- Do not touch or plug the power plug into an electrical outlet with wet hands.
- Do not use the chamber inside a bathroom, near a pool or other humid environments.



Precautions before and during use

Items may shift and move during shipping. When using the chamber for the first time, please **test the empty chamber without users** for 60 minutes first, to ensure the equipment is operating normally. If the equipment is operating abnormally, please do not use it and contact your ProReset hyperbaric advisor.

Warning: Never try to exit the chamber while it is under pressure. Before leaving the chamber, always open the manual pressure valve and wait for the pressure to drop until the gauge displays 0 kPa before opening the door of the chamber. Never forcibly open the door during use or when the depressurization has not completed yet as the door will get damaged during such misuse which is not covered under warranty.

In case of emergency or in a rare case when the manual pressure relief valve fails to work normally, please press **the emergency pressure relief valve** to reduce the pressure. When the pressure reaches 0 kPa, it is safe to open the door and exit the chamber.

When using the chamber for the first time and thereafter, **the ears may experience slight discomfort or pain** similar to when diving under water or flying in an airplane. This is a normal phenomenon caused by the different pressures in the eardrums. You can relieve this discomfort by chewing or swallowing saliva or blocking your nose with your thumb and forefinger and trying to blow air through your nose. After several times of use, you will get used to this or this phenomenon may gradually disappear.

Notice:


1. When the air pressure in the chamber rises and falls, the ears will have a squeezing sensation, just like the feeling of flying and lifting. This is normal. The following methods can be used for debugging: a. swallowing saliva; b. chewing gum; c.yawn. When the pressure in the chamber has reached its maximum and stabilizes at 2.0 ATA (100kPa), this discomfort will gradually disappear;



2. Before exiting , adjust the air pressure in the chamber through the pressure relief valve (Figure 0-1). Please make sure that the pressure gauge shows 0 (Figure 0-2) before you open the door;
3. After leaving the chamber, turn off the power supply of the 3 in 1 or the compressor, concentrator and chiller in a timely manner after the product is used, and clean and disinfect it in time.



(Figure 0-1)

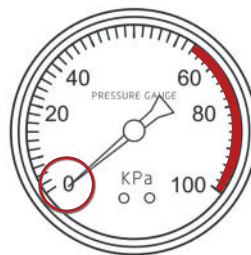


(Figure 0-2)

Specification structure





Specification structure

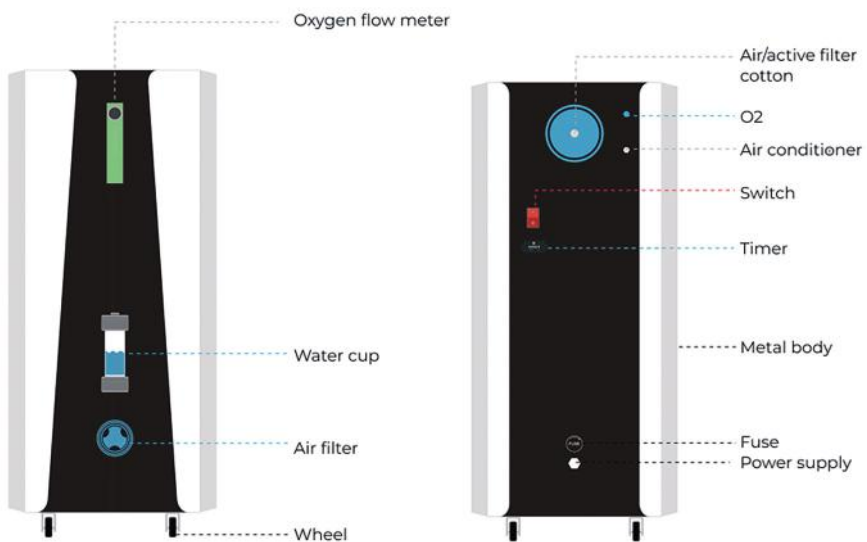




Specification structure

Vista		
Chamber	Pressure	1.2-2.0 ATA (adjustable)
	Material	Stainless steel material
	Size (LxWxH) / Weight	190 x 90 x 150 cm / 700 kg
3 in 1 (Compressor/O2 Concentrator/Cooler)	Features	All-in-one machine, a combination of air compressor, oxygen concentrator, and a cooler
	Material	Quite&Oil Free Type. Two super adsorption activate carbon filters in air Input and Outlet to protect. PM>2.5
	Size (LxWxH) / Weight	766 x 350 x 750 mm / 110 kg
	Air filter	Dual
	Power	1100W
	Voltage	120V/220V
	Flow of Air	72 l/min
	Flow of Oxygen	10 l/min, 93%+ oxygen
Or 3 separate units Air compressor+	Flow of Air	85 l/min
	Material	Quite&Oil Free Type. One super adsorption active carbon filter and one PE high-density super filter at Air inlet and Air outlet to protect.
	Air filter	Dual
	Size (LxWxH) / Weight	440 x 240 x 310 mm / 17 kg
	Power	320W
	Voltage	120V/220V
Oxygen Concentrator+	Flow of Oxygen	10 l/m, 93%+ oxygen
	Outlet pressure	≥135kPa ±3
	Features	1 set (2 pcs) of High-tech PSA molecular sieve (2 tower), for continuous oxygen production
	Noise level	≤60db
	Size (LxWxH) / Weight	650 x 400 x 365 mm / 31 kg
	Power	700W
Cooler	Voltage	120V/220V
	Flow of Air	150 l/min
	Outlet pressure	1.0MPa
	Size (LxWxH) / Weight	290 x 140 x 260 mm / 7 kg
	Power	210W
	Voltage	120V/220V



Product assembly and testing

Schematic installation

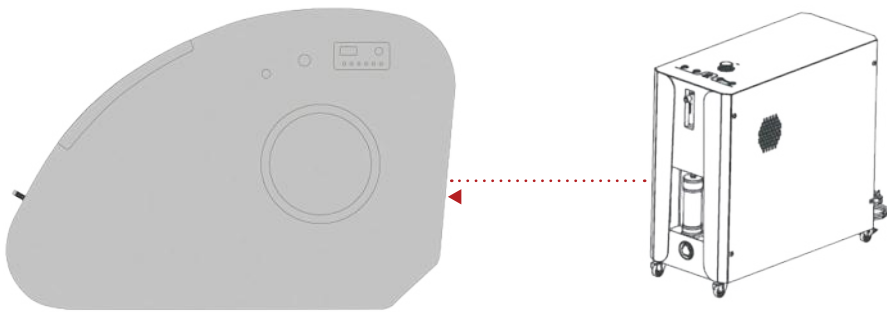

Unpacking

First, unpack everything.

The complete equipment includes two parts: the compressor (which contains the O₂ concentrator and the cooler) and the chamber.

Compressor assembly

Please follow the below steps carefully. Remove the two belts located underneath the compressor, pull them out.



At the back of the compressor, connect the two tubes to the corresponding ports at the back of the chamber. The black tube delivers airflow into the chamber at 80 liters/min, helping to remove carbon dioxide, excess oxygen, and regulate temperature for the user. The blue tube connects the O₂ concentrator, providing the user with 93% oxygen.



Note: Ambient air typically contains about 21% oxygen. With the O₂ concentrator, you will receive a higher dose of oxygen through the mask.

Next, turn on the compressor using the rear switch. On the top panel, press all the buttons (in any order). Make sure the blue valve is completely closed.

Note: On colder days, the "refrigerate" button may not be necessary.

Then connect the mask to the oxygen valve inside the chamber and ensure that the cable is neither bent nor damaged.

Important: Looking at the front of the compressor, you'll see a small ball that indicates the oxygen flow. It should always stay between 8 and 9. Check this before every session!

Pressure regulation

In case you would like to choose the pressure for your session, please use the manual pressure regulating valve.

Choose the pressure you want by rotating it, so it indicates the pressure you would like to be maintained.

If the pressure regulation valve is turned off so the indicator on the valve says OFF, then the pressure will increase to the maximum level automatically.





Product assembly and testing

How to adjust the seat



Ensure that there's no gap between the seat and your back.



When you decide to lie down, push with your back on the seat and the seat will automatically adjust to the position.



When you want to get in the upright position, start sitting down and the seat will automatically adjust.

How to enter and exit the chamber

Connect the power cord of the Air compressor to the power source, turn it on and run the equipment.




Turn on the power switch




Product assembly and testing


Before entering the chamber, open the manual pressure relief valve and make sure that pressure is at 0kPa.

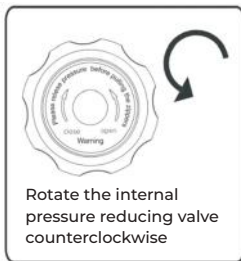

Enter the chamber, hold and close the door tightly by slowly and carefully sliding the door down without any exceeding force and close the manual pressure relief valve.

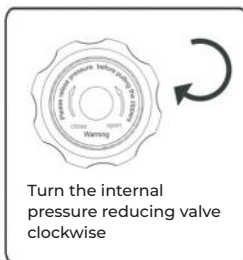

Carefully slide down door and gently push to close

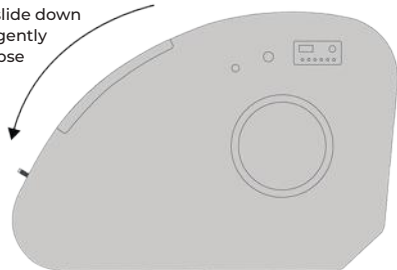


If the chamber door has been properly closed and the manual pressure relief valve turned clockwise to the closed position, you can press the ON button on the control panel. As the chamber pressurizes, the gauge will slowly start to rise to a maximum of 2.0 ATA (100kPa). Once maximum pressure has been achieved, the automatic pressure relief valve will open by themselves and start to work at this time to release excess pressure.




Product assembly and testing

WARNING: Use care to operate the door and always watch your or anyone else's fingers when opening and closing the door. Before leaving the chamber after use, make sure to depressurise the chamber by rotating the manual pressure-relief valve counter-clockwise to the open position to reduce pressure.

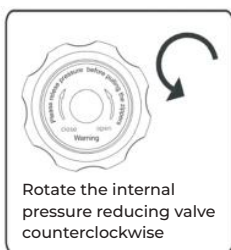

Gently pull and slide up door to open until held in place

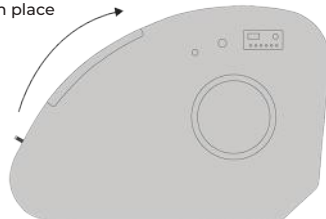


Ensure that the pressure gauge shows 0 kPa before you open the door gently and slide it up. Make sure the door is safely held in place when open before you get up and step out of the chamber.

After exiting the chamber, press the power switch to turn the 3in1 or the compressor, concentrator and chiller off and carefully slide the door down.

Use of communication equipment

Press the Call button on the control panel(1) to initiate a call. Please speak to the honeycomb speaker(2).

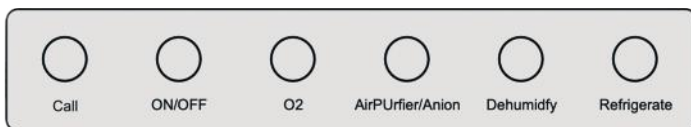

1




2



Product assembly and testing

Cooler/Chiller

The air output from the air compressor is condensed and dried by the built-in refrigerating dryer before the air flow enters the chamber body. This process may relatively remove the humidity from the air and collect it in the water cup.

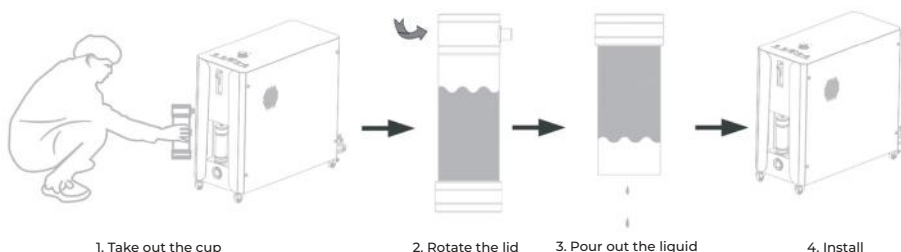

Notice: Please pay attention to the water level so it does not overflow. Speed of humidity collection will depend on the humidity level of your environment. It is recommended to drain the water cup after each use. After draining, when placing it back into the 3in1, make sure to align the humidity cup with the drainage port of the 3in1.

Console



The timer indicator should be at the "ON" position and the number is normally open. Turn the timer to the "OFF" position to turn it off.




Maintenance

Chamber maintenance and cleaning

You can clean the chamber body with a non-abrasive and very mild sterilizing spray or a 90% water 10% hydrogen peroxide solution on a soft microfiber cloth to wipe and dry the chamber, and then run the chamber idling for 20 minutes.

Maintenance and cleaning of Air compressor

It is recommended to replace the activated carbon filter element every 6 months, and please do it after the power switch has been pressed to the off position and the power cord has been unplugged.

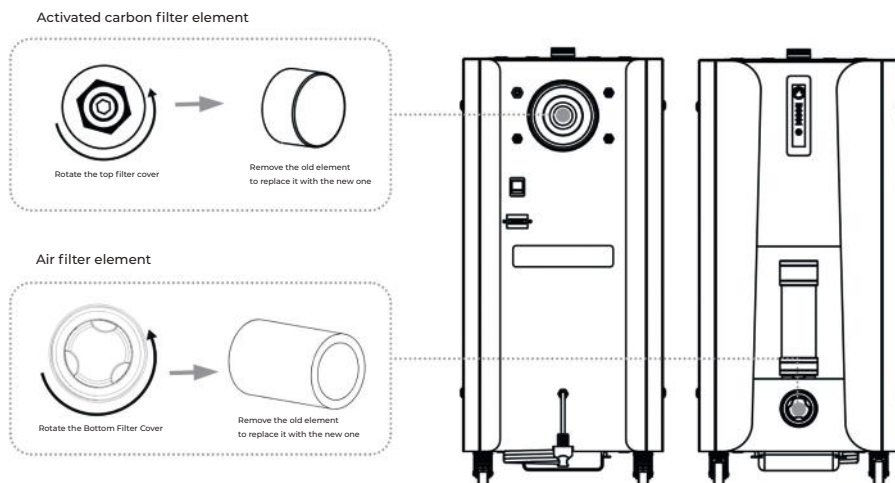



REMINDERS

CAUTION

REMEMBER THESE IMPORTANT STEPS

- ✓ To inflate the chamber turn the manual pressure relief valve to the close position
- ✓ To inflate the chamber be sure the door is completely closed and sealed
- ✓ Never open the door until the chamber has deflated and the gauge reads zero
- ✓ The compressor must be running during the whole time of the session
- ✓ Automatic pressure relief valve must release air when the chamber is at full pressure

CAUTION: Do not inflate the chamber with only pure oxygen; use the compressor provided. It is designed to work safely with the chamber.

CAUTION: Inspect all seams and fittings prior to use to ensure they are secure. Failure to do so could cause leaks reducing the performance of the chamber.

CAUTION: If any unusual noise or odor is detected, immediately discontinue use and contact the manufacturer. This may be an indication of a problem with the normal operation of the unit that needs to be addressed. In most situations, a simple call can resolve the issue.

WARNING: In the unlikely event that there is a rapid decompression, the individual inside the chamber must EXHALE. Please make sure that each individual using the chamber understands this precaution.

WARNING: Do not replace the compressor with any other model. The chamber is designed to work specifically with the 3in1 or compressor provided. For added safety, the compressor will not exceed the test pressure for the chamber.

WARNING: Do not tamper with the pressure regulation valves. These are factory set and there are no user adjustments. Any sign of tampering with these valves will void the warranty.



REMINDERS

CAUTION

REMEMBER THESE IMPORTANT STEPS

WARNING: Do not use the chamber in a manner where the chamber could roll off the surface causing a fall. Always use on a flat surface large enough to prevent rolling off and causing a fall.

WARNING: Do not use the electric 3in1 or compressor in the rain or around running water as this could cause electric shock.

WARNING: Wear comfortable clothing and lay on a mat, towel or blanket while in the chamber to limit contact with the chamber surface and to improve comfort and safety.

USE CARE to avoid the use of any sharp objects that may cause damage to the walls of the chamber.

CLEAN the inside of the chamber with a soft, damp microfiber cloth after each use.

INSPECT all hoses for leaks.

HUMIDITY – In humid climates, the chamber may create condensation. Simply dry with a soft towel.

EARS – Be sure to chew gum or swallow often during the pressurization.

ENSURE that all connections are tightly secured.

ENSURE the chamber is on a stable platform or on the floor.



ProReset Warranty Policy

ProReset warrants that its products are free from defects in materials and workmanship under normal and intended use. This 3-year limited warranty begins from the date of production and covers the original purchaser only. This warranty is non-transferable and requires proof of purchase for any claims. Annual maintenance, inspection and/or calibration services must be performed by ProReset or an authorized service provider to maintain warranty validity. Failure to comply will void this warranty.

Warranty Coverage

ProReset will, at its sole discretion, repair or replace defective products or components found to be faulty within the warranty period. Covered components include:

1. Chamber and its components
2. Compressor and its components

Parts not manufactured by ProReset but included in the product carry the original manufacturer's warranty.

Exclusions

This limited warranty does not cover:

1. Normal wear and tear.
2. Consumable parts, including air filters.
3. Shipping costs to and from ProReset's warehouse for repairs or replacements.
4. Damage incurred during transit (claims should be filed with the carrier).
5. Accessories or components added after shipment that are not approved by ProReset.
6. Products purchased through unauthorized dealers or third-party resellers.
7. External accessories not listed in ProReset's standard product catalog.
8. Yearly or other maintenance or service costs beyond warranty repairs.



ProReset Warranty Policy

Warranty Voiding Conditions

This warranty will be void if:

1. Cosmetic damage occurs not related to material or workmanship defects (e.g., scratches, dents, or tears).
2. Damage results from accidents, misuse, abuse, improper use or storage, or unauthorized modifications.
3. Repairs or alterations are performed without ProReset's prior written approval.
4. The product is used in conditions exceeding the guidelines outlined in the ProReset Operating Manual.
5. The serial number is removed, defaced, or untraceable.
6. The product is used with non-approved compressors, oxygen-enhancing devices, or accessories.
7. The Pressure Regulating Valves is tampered with or modified.
8. Cleaning agents containing harsh chemicals (e.g., acetone, paint thinner) are used.
9. The product is placed too close to heat sources, exposed to direct sunlight, or subjected to extreme environments.
10. The products is used outside the specified operational conditions.

Claim Process

- Claims must be submitted within 30 days of defect discovery and no later than the warranty expiration date.
- Provide proof of purchase and a detailed description of the defect with videos.
- Unauthorized repairs may render the product irreparable and void the warranty.
- Purchasers are responsible for shipping costs to and from ProReset's warehouse for warranty service.
- Claims should be submitted via certified mail or through the ProReset website.



ProReset Warranty Policy

Limitations of Liability

- ProReset's liability is limited to repair or replacement at its discretion under this warranty.
- No compensation is provided for incidental damages, loss of use, downtime, or consequential losses.
- Purchasers assume full responsibility for compliance with local safety codes and operational regulations.

General Conditions

- This warranty applies only to products used and maintained per ProReset's instructions and specifications.
- Resale, rental, lease, or any transfer of ownership voids this warranty.
- No implied warranties of merchantability or fitness for a particular purpose extend beyond this document.



Troubleshooting

If your device is not functioning properly, please refer to the troubleshooting and solutions in the following list. If the problem still persists please contact your ProReset hyperbaric advisor.

Failure phenomenon	Cause	Solution
Chamber pressure boost is slow	1. The air compressor connector or air intake connector falls off or cracks. 2. The air filter cotton is full of dust. 3. The air inlet is blocked and cannot be discharged. 4. The air compressor screw are loose 5. The manual pressure relief valve is not closed all the way.	1. Make sure that there is no blockage or parts falling off, reconnect the hoses. 2. Clean or replace the filter cotton. 3. Clean the air inlet of the air compressor. 4. Tighten the screws on the air compressor. Notice: Do not overtighten. 5. Tighten the manual pressure relief valve. Notice: Do not use excessive force and overtighten the valve as it may break.
The pressure can't go up	1. Did not properly close the door. 2. The manual pressure relief valve is not closed or tightened. 3. The air pipe is not properly connected.	1. First release the chamber pressure, then close the door tightly without using excessive force and restart. 2. Gently close the manual pressure relief valve all the way to the close position. 3. Find the source of the air leak and reconnect the air tube on the chamber or air compressor. Inside air compressor.
Abnormal noise or vibration inside Air compressor	1. The air filter is clogged 2. The air intake of the air compressor is blocked, no air enters. 3. The air compressor screws are loose.	1. Make sure that the air filter is not damaged or replace the filter. 2. Clean the air inlet of the air compressor. 3. Tighten the screws of the air compressor.

If you have any questions about your product
please feel free to contact us:

ProReset® S.L
C/Londres 18 - P.I. Mas de Tous
La Pobla de Valbona
Valencia 46185 Spain
+34 624 198 129
support@proreset.com
www.proreset.com